




85 Portland Avenue, Bergenfield, NJ 07621

800-999-4950 www.equipoisedental.com

PROPER USAGE OF THE EQUIPOISE ® ANALYZER

The ANALYZER

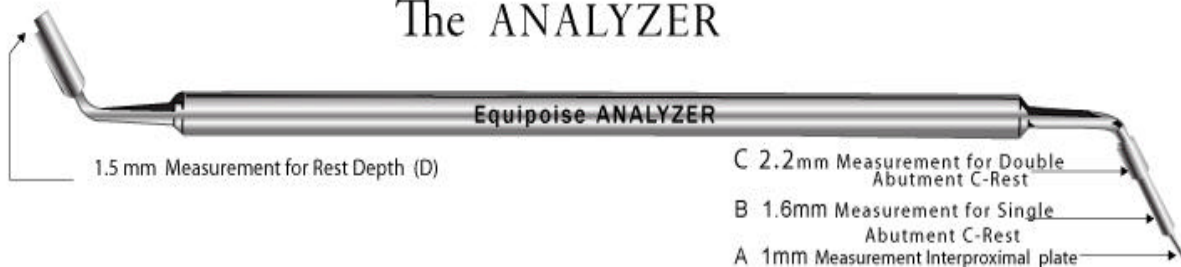

1. 1MM IN DIAMETER

This diameter is used to measure the space created for the interproximal plate.

2. 1.6MM IN DIAMETER

This diameter is used to measure the space created for the C-rest when single abutments are to be utilized.

This rung of the analyzer is also ideal for measuring occlusal reduction when preparing teeth for PFM crowns.

3. 2MM IN DIAMETER

This diameter is used to measure the space created for the interproximal C-rest when double abutments are utilized.

4. 1.5MM IN LENGTH

This length is used to measure depth of rest preparations for an Equipoise partial denture.